


AQUA-PRESS® Sterile Irrigation System

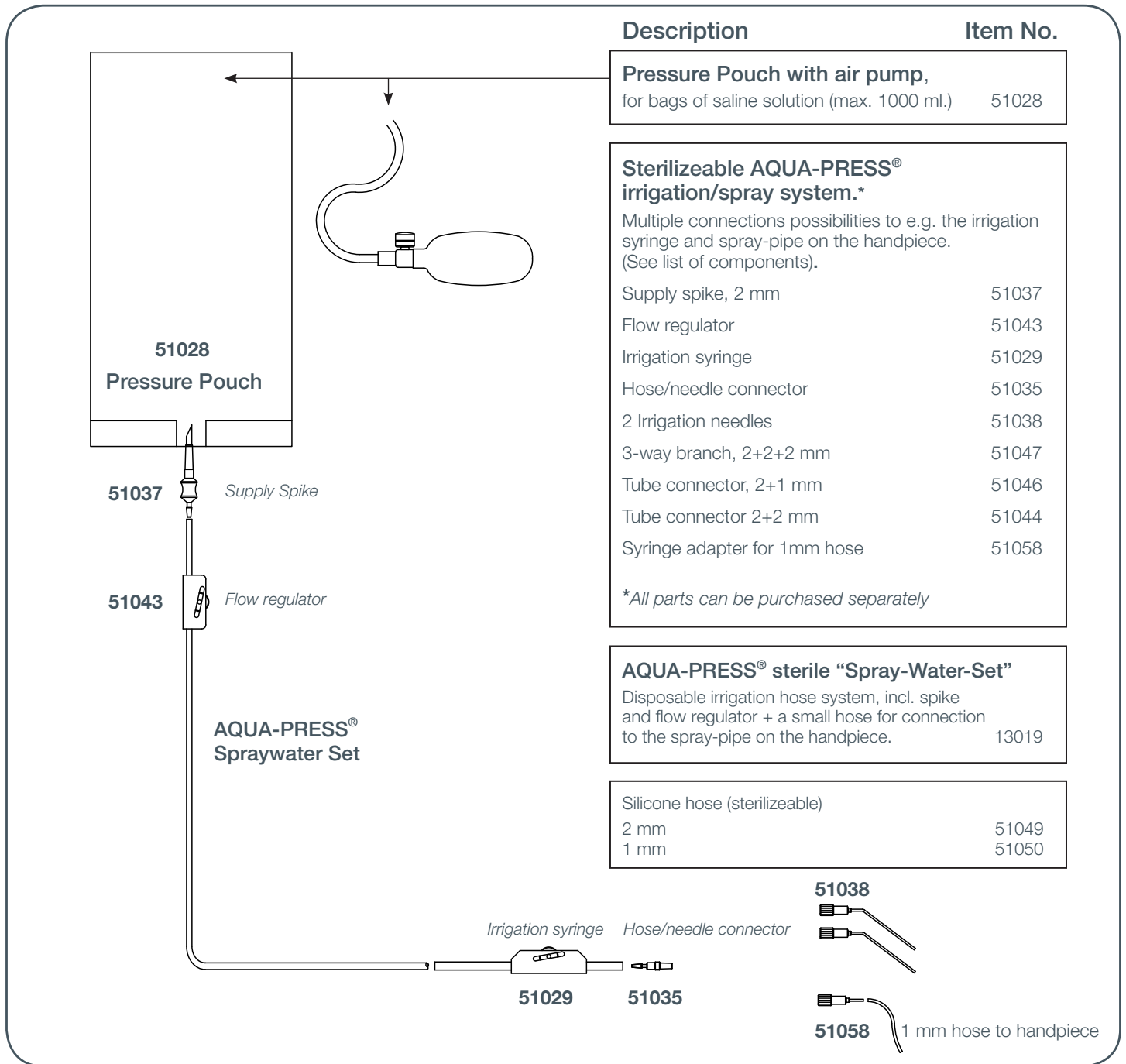

How to set up and connect the AQUA-PRESS® system.

1. Cut the desired length of the 2 mm (51049) silicone hose.
2. Place the flow regulator (51043) onto the hose.
3. Place the supply spike (51037) at the end of the hose.
4. **Assembling of the irrigation syringe (51029):**
Introduce the hose all the way through the syringe, place the "needle connector" (51035) onto the hose and pull back the hose until the needle connector is securely fixed onto the syringe. Attach an irrigation needle (51038) to the connector (51035).
5. Connect the supply spike (51037) to the bag of saline solution. NB: Make sure that the flow has been turned off (flow regulator (51043) closed).
6. AQUA-PRESS® is ready for use when the pressure pouch is inflated. The pressure pouch can be hanged on a hook or simply laid down on a table in a convenient distance of the operation site.

By fitting the syringe adapter (51058) to the irrigation syringe, the AQUA-PRESS® system can be connected to the external or internal cooling attachment of the surgical handpiece by using a piece of 1mm silicone hose (51050). The 3-way branch (51047) and the 2+1mm connector (51046) also included in the set provide further options of individual set-up.

Cleaning and sterilization of the AQUA-PRESS® system

It is highly recommended to carefully rinse through the tube system, syringe and needle in tap water shortly after use and before sterilization in order to avoid discoloration of the metal parts.
Hoses and metal parts can be sterilized by 135°C.

INSTRUCTIONS

AQUA-PRESS®

STERILE IRRIGATION SYSTEM



RØNVIG Dental Mfg. A/S

Gl. Vejlevej 59 • DK-8721 Daugaard • Tel.: +45 70 23 34 11
Fax: +45 76 74 07 98 • E-mail: export@ronvig.com • www.ronvig.com