



# CLINICAL PROCEDURE

Clean the tooth surface

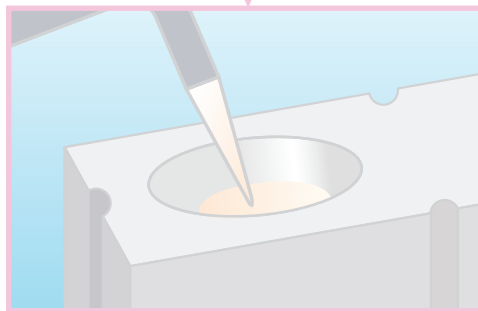

Saturate Felt Applicator with MS Coat ONE

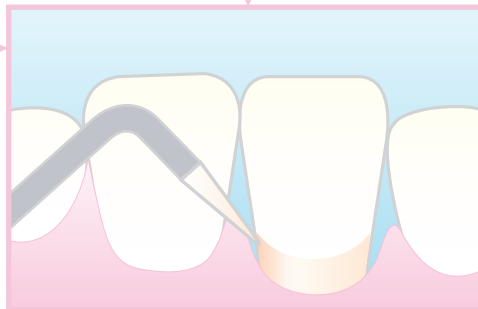

Apply MS Coat ONE with the Felt Applicator for 30 seconds using a pumping / rubbing motion



Dry with air-syringe

Rinse

Completed

If the surface remains painful, you can repeat the treatment (rub with the saturated applicator and air-dry) 2 or 3 times at the same appointment

Contents of the kit			
1. MS Coat ONE .....	5 ml	3. Felt Applicator .....	100
2. Felt Holder .....	1	4. Plastic Dispensing Dishes...	20

Exclusive distributor



**GENERIQUE INTERNATIONAL**

77/79 rue Pierre BROSOLETTÉ • 93160 NOISY LE GRAND  
Tel. +33 1 43 03 06 84 • Fax +33 1 43 03 15 98

E-mail : generinter@aol.com - Web : www.generiqueinternational.com

Made in Japan by



571-2 Furutaka-cho, Moriyama,  
Shiga, 524-0044, Japan  
Phone:81-77-582-9981 Fax:81-77-582-9984  
<http://www.sunmedical.co.jp>