



FKG
swiss endo



NiTi rotary files
for mechanised
glide path preparation

Race ISO 10

Designed for calcified or narrow canals

► The FKG Race range

An internationally-recognised reference, Race (Reamer with Alternating Cutting Edges) is the most complete selection of NiTi rotary files for root canal treatment.

Race ISO 10

Instruments for mechanised glide path preparation of calcified or narrow canals

3 instruments make up the Race ISO 10 range, all with a size of ISO 10 and with .02, .04 and .06 taper. This represents a significant technological advance. The Race ISO 10 are designed to reach working length in calcified or very narrow canals when manual K files of ISO 06 or 08 cannot progress further. They turn at a speed of 600-800 rotations per minute. The Race ISO 10 files enable the

faster and safer creation of a glide path than with manual instruments. Thanks to the Race ISO 10 range's slender design and their exceptional flexibility, the instruments follow the canal's anatomy perfectly, even in the median and apical third. They are used without pressure up to working length. Root canal treatment may then be carried out with the iRace or BioRace ranges.

Illustration of a case with calcified/obliterated and narrow canals

Courtesy of Dr. Gilberto Debelian (Norway)



Pre-operation
Diagnosis: pulpitis



Difficulty:
Narrow, calcified



Post-operation
MB & DB - 35/.04
P - 50/.04


FKG Race: Specifications

Several exclusive technological advances lie behind Race range's efficacy and worldwide renown:

- ▶ Exclusive rounded safety tips for precise guiding
- ▶ Alternating cutting edges to eliminate screwing-in effect
- ▶ Sharp edges for best cutting efficiency
- ▶ Electro-chemical polishing for better resistance to torsion and fatigue
- ▶ SafetyMemoDisc (SMD) to master metal fatigue

Race ISO 10 availability

Individual Files	5 pieces/box
Assortment	2 × 10/02, 2 × 10/04, 2 × 10/06
Lengths	21 mm, 25 mm, 31 mm

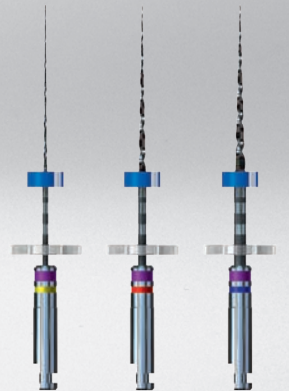

ISO 10/02 ISO 10/04 ISO 10/06



FKG Dentaire SA

Founded in 1931, FKG Dentaire SA is today at the very cutting edge of development, production and distribution of dental products for dentists, endodontists and laboratories. Precision has always been central to FKG, and the company naturally made its base in «Watch Valley», home to most Swiss watchmaking and microtechnical companies.

FKG was given a new lease of life in 1994, when Jean-Claude Rouiller took up the reins. A CEO with a vision, he set out a strategy based on innovative products and the concept of machinery specially manufactured for the dental industry. Mr Rouiller also enlarged the

distribution network to encompass more than 80 countries worldwide. FKG products are certified according to international norms and standards.

The Swiss Venture Club named FKG «Western Switzerland Company of the Year 2012», a reward for the company's dynamism, high product quality, and its continuing innovation.

**« Western
Switzerland
Company of the
Year 2012 »**



FKG Dentaire SA, Crêt-du-Loche 4, CH-2304 La Chaux-de-Fonds, Switzerland
T +41 32 924 22 44, F +41 32 924 22 55, info@fkg.ch, www.fkg.ch