

FLEXI TRIX™ CLAMP™

For Class II composite restorations




For tight and
anatomically
correct contact
points

FLEXI-TRIX™

The Pre-contoured Sectional Matrix Band
for Class II Composite Restorations

- Easy to contour - no metal "memory"
- Slightly pre-contoured
- Non-stick smooth surface
- 50 µ thick, 3 sizes
- Easy to place, also subgingivally
- Does not collapse when placed

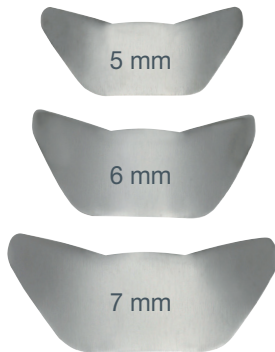

Material: Non retentive smooth stainless steel

FLEXI-CLAMP™

The Hygienic user friendly Clamp System
for Class II Composite Restorations

- Efficient matrix band fixation
- Stable when placed
- No risk of pop-off tooth injuries
- Hygienic, designed for single use

MORE THAN JUST A RING







Material: Engineering polymer

HOW TO USE

FLEXI-TRIX™ & FLEXI-CLAMP™

Controlled tooth separation is achieved by the use of a correctly fitting wedge and an appropriate force on the wedge.

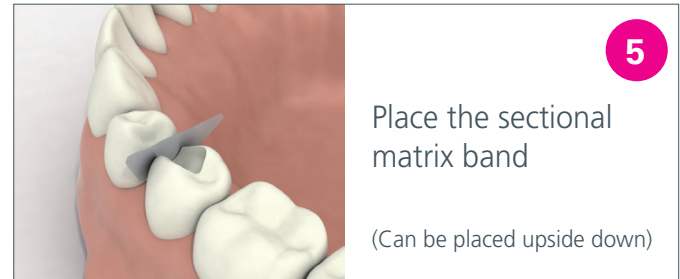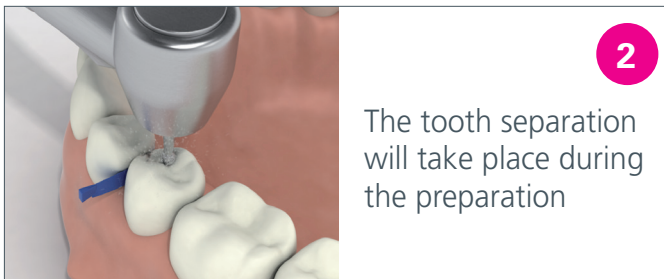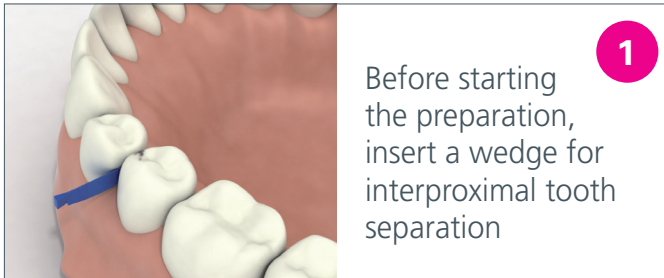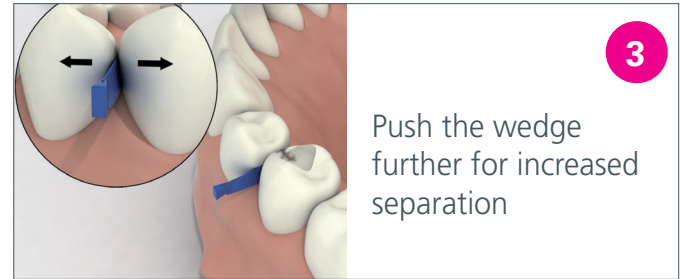

6


Insert a wedge

7

Place the
FLEXI-CLAMP

8

Contour the matrix
band to approximal
contact

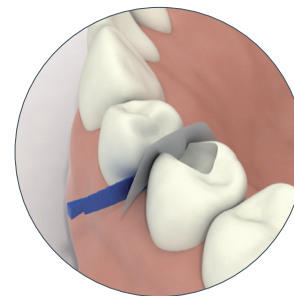

TIP

During preparation,
FLEXI-TRIX can be used to
protect the adjacent tooth.



WARNING!

FLEXI-CLAMP™ is a disposable "single use clamp" made from a high strength engineering polymer material. High temperature or chemical cleaning/sterilization of the clamp can cause stress in the material, risking the clamp to crack.

The manufacturer cannot assume any liability for damages or bodily harm if the FLEXI-CLAMP™ is recycled for repeated use.



View product animation



RØNVIG
Dental Mfg. A/S

Gl. Vejlevej 59 . DK-8721 Daugaard
Tlf: +45 70 23 34 11. Fax: +45 76 74 07 98
email: export@ronvig.com. www.ronvig.com